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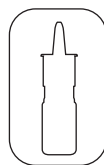
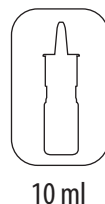
ACTIVE SPRAY

Components.

Active components: European cyclamen dry extract – 2.6% (liquid extract – 41.6%).

Auxiliary components: water, carbomer, sodium hydroxide, methyl paraben, sodium benzoate, sodium thiosulfate.

Description.

Gel-like mass of light yellow to yellow-brown colour, without foreign inclusions. A slightly grainy texture is allowed.

Form of manufacture.

Spray in a polymer bottle with a volume of 10 ml.

Properties.

The effect of the nasal spray “SINUS LIFT ACTIVE SPRAY” is due to the properties of the components that make up the composition.

The main intended effect of nasal spray “SINUS LIFT ACTIVE SPRAY” in or on the human organism is not achieved by pharmacological, immunological or metabolic means.

European cyclamen extract helps to restore comfortable nasal breathing due to its complex effect on the nasal mucosa. The effectiveness of the component is ensured by pharmacologically active substances in its composition – highly purified saponins.

They contribute to:

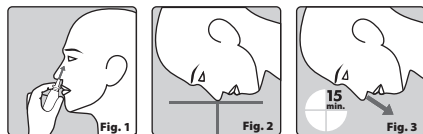
- restoring the patency of the sinuses;
- eliminate inflammation and swelling;
- normalisation of water-salt balance of the mucous membrane.

Carbomer acts as a thickener and gelling agent.

Sodium thiosulfate is used as an antioxidant.

Nasal spray “SINUS LIFT ACTIVE SPRAY” is used:

- to cleanse and protect the physiological functions of the mucous membrane of the sinuses in purulent and inflammatory processes;
- to restore comfortable nasal breathing and functions of the nasal mucosa, impaired as a result of pyoinflammatory changes in the paranasal sinuses and active therapeutic measures.

Method of application.

In each nasal passage, make 1-2 sprays of the spray (Fig. 1). After 1-2 minutes, it is recommended to tilt the head down for 10-15 minutes so that the tip of the nose is the lowest point of the head position (Fig. 2), and during this time slowly blow out both halves of the nose without pinching the nose (Fig. 3). With this head position, the contents of all sinuses are emptied, so significant nasal discharge is possible. The procedure should be performed no more than 2 times a day. Children from 6 to 12 years of age may use the product after consulting a doctor.

Features of application.

The course of use is not more than 7 days. During use,

there may be a short-term feeling of mild burning in the nasopharynx, reflex phenomena in the form of salivation, and rarely short-term lacrimation. In none of these cases is there a need to discontinue use.

AVOID CONTACT OF NASAL DROPS WITH EYES. DO NOT INHALE INTENSIVELY WHILE ADMINISTERING NASAL DROPS.

This product can be used by children from the age of 12 years.

Children should be used only in the presence of an adult. Nasal spray “SINUS LIFT ACTIVE SPRAY” do not affect the speed of psychomotor reactions.

To prevent the spread of infection, do not share one bottle with more than one person.

Do not use this product in persons with individual hypersensitivity to its components.

Expiration date.

See on the packaging. After opening the bottle, it is recommended to use the product within 3 months.

Storage conditions.

Store at a temperature between +5 °C and +25 °C in a dark place and out of the reach of children.

Do not use after the expiry date indicated on the package.

To prevent counterfeiting, the original packaging of the product contains an insert with a unique code in addition to the instructions for use.

Manufacturer.

“Georg BioSystems” LLC.

3, Vokzalna str., Pobuzke village, Holovanivsk district, Kirovohrad region, 26555, Ukraine.

Address of manufacture.

64B, Dorozhna str., city of Kamianske, Dnipropetrovsk region, 51905, Ukraine.

Tel.: 0 800 501 161. **E-mail:** contact@georgbio.com

www.georgbio.com

THE MEANING OF THE SYMBOLS SHOWN ON THE MARKING:

	Use – by date
	Batch code
	Manufacturer
	Date of manufacture
	Temperature limit
	Consult instructions for use
	Mark of conformity with technical regulations and identification number of the designated conformity assessment authority
	Keep away from sun light

To prevent counterfeiting, the original packaging of the product contains an insert with a unique code in addition to the instructions for use.

Side 1	Side 2
	AA 00 00 00 00 00

Date of the last version of the instructions for use: 31.07.2024