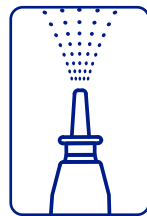
UA.TR.120

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Instructions for use nasal spray "CHISTONOS SPRAY"

30 ml

Components.

Active components: sea salt – 0.7%, aloe extract – 0.16%, calendula extract – 0.16%.

Auxiliary components: water, sodium benzoate, citric acid, D-panthenol, furacilin, β-carotene.

Description.

A homogeneous liquid from light yellow to yellow in color, without foreign inclusions and impurities.

Form of manufacture.

Spray in a polymer bottle with a capacity of 30 ml. Not less than 220 doses.

Properties.

Action of "CHISTONOS SPRAY" nasal spray is determined by properties of the components included in its composition.

The main intended effect of "CHISTONOS SPRAY" nasal spray in human body is not achieved by pharmacological, immunological or metabolic means.

Isotonic solution of sea salt facilitates the normal physiological state of the nasal mucous membrane, reduces mucosal swelling, improves sinus drainage due to natural physiological processes. It improves the ciliate epithelium function, which strengthens protection functions of the mucous membrane of the nasal cavity and paranasal sinuses against penetration of pathogenic bacteria and viruses.

β-carotene is used as an antioxidant.

Calendula extract provides anti-inflammatory, antimicrobial and wound-healing effects due to the enhancement of antioxidant defense mechanisms. It releases irritation and soothes the mucous membranes.

Aloe extract has an antibacterial and anti-inflammatory effect. It regulates the acid-alkaline balance of mucous membranes. It contains vitamins, minerals, enzymes and essential amino acids due to which it works as a biogenic stimulant.

D-panthenol binds the moisture content in the nasal mucosa and thus the sea water effect is optimally maintained. During its stay in the body, it turns into pantothenic acid (vitamin B5), which promotes regeneration and healing, and also promotes the activation of mucosa cells.

Furacilin is included in the composition as an auxiliary substance in order to prevent microbial degradation of the product, to maintain the number of microorganisms at a low and safe level.

"CHISTONOS SPRAY" nasal spray is used:

- for the care of the mucous membrane of the nasal sinuses in case of purulent-inflammatory changes in the nasal cavity restore and protect the physiological functions of the mucous membrane;
- for prevention of infections of the nasal cavity, including influenza;
- by persons suffering from nasal breathing problems, dryness of the nasal mucosa, presence of crusts in the nose (air-conditioned and heated premises), smokers, motor vehicle drivers, people working in hot and dusty workshops, in order to restore the physiological functions of the nasal mucosa.

Method of application.

To restore the functions of the nasal mucosa in acute and chronic disorders of nasal breathing – 4-6 times per day, 1-2 injections into each nasal passage.

For prevention of infections – 4-6 times per day, 2 injections into each nasal passage.

To soften and remove nasal secretions, use 2-3 times per day by means of 2 injections in each nasal passage, until accumulations of nasal secretions deposited in the nasal passage are softened and removed.

Features of application.

Usage by children is allowed only in the presence of adults.

"CHISTONOS SPRAY" nasal spray does not affect the speed of psychomotor actions.

In order to prevent spread of infection, one bottle should not be used by several people.

This product must not be used by people with individual hypersensitivity to its components.

Expiration date.

See on the packaging. After opening the bottle, it is recommended to use the product within 6 weeks.

Storage conditions.

Store at a temperature from +5 °C to +25 °C in a place protected from light and inaccessible to children.

Do not use after expiration of the date indicated at the package.

Manufacturer.

"Georg BioSystems" LLC.

3, Vokzalna str., Pobuzske village, Holovanivsk district, Kirovohrad region, 26555, Ukraine.

Address of manufacture.

64B, Dorozhna str., city of Kamianske, Dnipropetrovsk region, 51905, Ukraine.

Tel.: 0 800 501 161. **E-mail:** contact@georgbio.com

www.georgbio.com

THE MEANING OF THE SYMBOLS SHOWN ON THE MARKING:

	Use – by date
	Batch code
	Manufacturer
	Date of manufacture
	Temperature limit
	Consult instructions for use
	Mark of conformity with technical regulations and Identification number of the designated conformity assessment body
	Keep away from sun light

To prevent counterfeiting, the original packaging of the product contains an insert with a unique code in addition to the instructions for use.

Side 1	Side 2
	AA 00 00 00 00 00