UA.TR.120

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Instructions for use nasal spray "CHISTONOS SPRAY"

Components.

Active components: sea salt – 0.7%, aloe extract – 0.16%, calendula extract – 0.16%.

Auxiliary components: water, sodium benzoate, citric acid, D-panthenol, furacilin, β -carotene.

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Form of manufacture.

Spray in a polymer bottle with a capacity of 30 ml. Not less than 220 doses.

Properties.

Action of "CHISTONOS SPRAY" nasal spray is determined by properties of the components included in its composition.

The main intended effect of "CHISTONOS SPRAY" nasal spray in human body is not achieved by pharmacological, immunological or metabolic means.

Isotonic solution of sea salt facilitates the normal physiological state of the nasal mucous membrane, reduces mucosal swelling, improves sinus drainage due to natural physiological processes. It improves the ciliate epithelium function, which strengthens protection functions of the mucous membrane of the nasal cavity and paranasal sinuses against penetration of pathogenic bacteria and viruses.

β -carotene is used as an antioxidant.

Calendula extract provides anti-inflammatory, antimicrobial and wound-healing effects due to the enhancement of antioxidant defense mechanisms. It releases irritation and soothes the mucous membranes.

Aloe extract has an antibacterial and anti-inflammatory effect. It regulates the acid-alkaline balance of mucous membranes. It contains vitamins, minerals, enzymes and essential amino acids due to which it works as a biogenic stimulant.

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Furacilin is included in the composition as an auxiliary substance in order to prevent microbial degradation of the product, to maintain the number of microorganisms at a low and safe level.

"CHISTONOS SPRAY" nasal spray is used:

- for the care of the mucous membrane of the nasal sinuses in case of purulent-inflammatory changes in the nasal cavity restore and protect the physiological functions of the mucous membrane;
- for prevention of infections of the nasal cavity, including influenza;
- by persons suffering from nasal breathing problems, dryness of the nasal mucosa, presence of crusts in the nose (air-conditioned and heated premises), smokers, motor vehicle drivers, people working in hot and dusty workshops, in order to restore the physiological functions of the nasal mucosa.

Method of application.

To restore the functions of the nasal mucosa in acute and chronic disorders of nasal breathing – 4-6 times per day, 1-2 injections into each nasal passage.

For prevention of infections – 4-6 times per day, 2 injections into each nasal passage.

To soften and remove nasal secretions, use 2-3 times per day by means of 2 injections in each nasal passage, until accumulations of nasal secretions deposited in the nasal passage are softened and removed.

Features of application.

Usage by children is allowed only in the presence of adults.

"CHISTONOS SPRAY" nasal spray does not affect the speed of psychomotor actions.

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
UA.TR.120

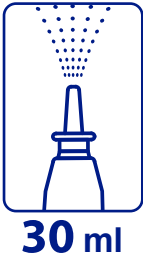Instructions for use nasal spray “CHISTONOS SPRAY FOR CHILDREN”

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Properties.

Action of “CHISTONOS SPRAY FOR CHILDREN” nasal spray is determined by properties of the components included in its composition.

The main intended effect of “CHISTONOS SPRAY FOR CHILDREN” nasal spray in human body is not achieved by pharmacological, immunological or metabolic means.

Isotonic solution of sea salt facilitates the normal physiological state of the nasal mucous membranes, reduces mucosal swelling, improves sinus drainage due to natural physiological processes. It improves the ciliate epithelium function, which strengthens protection functions of the mucous membrane of the nasal cavity and paranasal sinuses against penetration of pathogenic bacteria and viruses.

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Calendula extract provides anti-inflammatory, antimicrobial and wound-healing effects due to the enhancement of antioxidant defense mechanisms. It releases irritation and soothes the mucous membrane.

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“CHISTONOS SPRAY FOR CHILDREN” nasal spray is used:

- for the care of the mucous membrane of the nasal sinuses in case of purulent-inflammatory changes in the nasal cavity to restore and protect the physiological functions of the mucous membrane;
- for prevention of infections of the nasal cavity, including influenza;
- by children suffering from problems with nasal breathing, dryness of the nasal mucosa, presence of crusts in the nose, in order to restore physiological functions of the nasal membrane.

Method of application.

To restore functions of the nasal mucous membrane in case of acute and chronic disorders of nasal breathing:

- children from 3 years old – on the recommendation of a doctor;
- children from 6 to 9 years old – 2-4 times per day;
- children from 9 to 12 years old – 4-6 times per day, 2 injections.

To prevent infections:

- children from 3 years old – on the recommendation of a doctor;
- children from 6 to 9 years old – 2 times per day;
- children from 9 to 12 years old – 3-4 times per day, 1 injection.

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Features of application.

This product can be used by children aged 6 and older.

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“CHISTONOS SPRAY FOR CHILDREN” nasal spray does not affect the speed of psychomotor actions.

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Instructions for use nasal spray “CHISTONOS SPRAY. EFFECTIVE RINSING”

Components.

Active components: sea salt – 0.7%, calendula extract – 0.16%.

Auxiliary components: water, sodium benzoate, citric acid, D-panthenol, furacilin.

Description.

A homogeneous liquid from light yellow to yellow in color, without foreign inclusions and impurities.

Form of manufacture.

Spray in a polymer bottle with a capacity of 120 ml. Not less than 500 doses.

Properties.

The action of “CHISTONOS SPRAY. EFFECTIVE RINSING” nasal spray is determined by properties of the components included in its composition.

The main intended effect of “CHISTONOS SPRAY. EFFECTIVE RINSING” nasal spray in human body is not achieved by pharmacological, immunological or metabolic means.

Isotonic solution of sea salt facilitates the normal physiological state of the nasal mucous membranes, reduces mucosal swelling, improves sinus drainage due to natural physiological processes. It improves the ciliate epithelium function, which strengthens protection of the mucous membrane of the nasal cavity and paranasal sinuses against penetration of pathogenic bacteria and viruses.

D-panthenol binds the moisture content in the nasal mucosa and thus the sea water effect is optimally maintained. During its stay in the body, it turns into pantothenic acid (vitamin B5), which promotes regeneration and healing, and also promotes the activation of mucosa cells.

Calendula extract has anti-inflammatory, antimicrobial and wound-healing effects due to the enhancement of antioxidant defense mechanisms. It reduces irritation and soothes the nasal mucous membrane.

Furacilin is included in the composition as a preservative in order to prevent microbial degradation of the product, to maintain the number of microorganisms at a low and safe level during the shelf life.

“CHISTONOS SPRAY. EFFECTIVE RINSING” nasal spray is used:

- as an auxiliary hygienic-prophylactic agent in the complex therapy of acute and chronic purulent-inflammatory processes for the physiological care of the mucous membrane of the nasal sinuses and after surgical interventions in the nasal cavity to restore the protective functions of the mucous membrane;
- for prevention of nasal cavity infections associated with colds, including influenza;
- for cleaning the nasal mucous membrane from viruses, bacteria, allergens, dust;
- by persons suffering from nasal breathing problems, dryness of the nasal mucosa, presence of crusts in the nose, with the aim of normalizing physiological functions and moisturizing the nasal mucosa;
- to provide additional protection of adults and children from bacteria and viruses in periods associated with increased risk of diseases and epidemics, including when visiting places of crowding and organized children's groups.

Method of application.

The spray is used intranasally. Before the first use of the spray, you should press the dispenser several times until a stable spray appears.

To restore the physiological functions of the mucous membrane in case of acute and chronic disorders of nasal breathing:

- children from 3 years old – on the recommendation of a doctor;
- children from 6 years old and adults – 3-6 times per day, 1 injection in each nasal passage.

To prevent infections:

- children from 3 years old – on the recommendation of a doctor;
- children from 6 years old and adults – 1-3 times per day, 1 injection in each nasal passage.

To rinse the nasal cavity from secretions, use it for children from 3 years old (on the recommendation of a doctor), children from 6 years old and adults 2-3 times per day, 1 injection into each nasal passage, until accumulations of nasal secretions deposited in the nasal passage is softened and removed.

The procedure for using the nasal spray

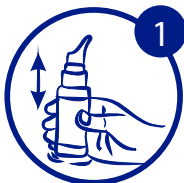
I. Preparation:

- clean the nasal passages thoroughly;
- shake the bottle vigorously;
- remove the safety cap from it;
- make sure the dispenser is well connected.

II. For irrigation of the nasal mucosa:

- holding the bottle at any angle, carefully insert the tip into the nasal passage and press the dispenser once;
- while injecting, take a light breath through the nose at the same time;
- repeat the procedure with the other nasal passage;
- clean the nose from secretions by means of carefully exhaling air through it;
- wash the dispenser tip with clean water, wipe it dry and close the safety cap.

III. For rinsing the nasal cavity: the procedure can be carried out both standing and sitting, preferably before eating. To do this, repeat the actions shown in the pictures.

Get prepared for the procedure of rinsing the nose, as indicated in section I.

Tilt your head to the left and insert the spray tip into the right nasal passage. Hold your breath and press the dispenser once in order to make an injection.

Tilt your head to the right and insert the spray tip into the left nasal passage. Hold your breath and press the dispenser once in order to make an injection.

Do not tilt your head back to avoid the feeling of discomfort caused by accidental liquid entering the throat.

After rinsing gently blow the nasal passages to free them of secretions.

Wash the dispenser tip with clean water, wipe it dry and close the safety cap.

Features of application.

This product can be used by children aged 6 and older. Usage by children is allowed only in the presence of adults.

The action of “CHISTONOS SPRAY. EFFECTIVE RINSING” does not affect the speed of psychomotor actions.

In order to prevent spread of infection, one bottle should not be used by several people.

This product must not be used by people with individual hypersensitivity to its components.

Expiration date.

See on the packaging. After opening the bottle, it is recommended to use the product within 6 weeks.

Storage conditions.

Store at a temperature from +5 °C to +25 °C in a place protected from light and inaccessible to children.

Do not use after expiration of the date indicated at the package.

Manufacturer.

“Georg BioSystems” LLC.
3, Vokzalna str., Pobuzke village, Holovanivsk district,
Kirovohrad region, 26555, Ukraine.

Address of manufacture.

64B, Dorozhna str., city of Kamianske, Dnipropetrovsk region, 51905, Ukraine.

Tel.: 0 800 501 161.

E-mail: contact@georgbio.com

www.georgbio.com

THE MEANING OF THE SYMBOLS SHOWN ON THE MARKING:

	Use – by date
	Batch code
	Manufacturer
	Date of manufacture
	Temperature limit
	Consult instructions for use
	Mark of conformity with technical regulations and Identification number of the designated conformity assessment body
	Keep away from sun light

To prevent counterfeiting, the original packaging of the product contains an insert with a unique code in addition to the instructions for use.

Side 1	Side 2
	AA 00 00 00 00 00